



# **TOTAL HYGIENE SOLUTION**



**Clinical**  
*Disinfectants*





# ***zeta hygiene***

***Broad spectrum,  
rapid action disinfection***



ZHERMACK's range of disinfection and sterilisation products satisfies the needs of the modern dental surgery.



**Disinfection of surgical and rotating instruments**  
zeta 1 ultra - zeta 2 sporex - zeta 2 enzyme



**Surface disinfection of medical devices**  
zeta 3 ultra - zeta 3 soft - zeta 3 wipes - zeta 3 wipes TOTAL



**Delicate surface disinfection of medical devices**  
zeta 3 foam - zeta 3 wipes POP-UP



**Deterision of washable surfaces**  
zeta 4 wash



**Disinfection of aspiration circuit**  
zeta 5 unit



**Deterision of hands**  
zeta 6 hydra - zeta 6 drygel



**Disinfection of impressions**  
zeta 7 solution - zeta 7 spray



○ SURFACE DISINFECTION OF MEDICAL DEVICES

## zeta 3 ultra

○ Marine fragrance



### APPLICATIONS

**ZETA 3 ULTRA** is an aldehyde-free, ready-to-use full spectrum disinfectant and detergent, developed and tested according to the latest harmonised European disinfection standards.

**ZETA 3 ULTRA** has been designed for rapid, high level disinfection and cleaning between patients of medical device surfaces (handpieces, contra-angles, etc.)\*.

#### SPECTRUM

**Bactericidal:** EN 13727, EN 14561 (*S. aureus*, *P. aeruginosa*, *E. hirae*)

**Fungicidal:** EN 13624, EN 14562 (*C. albicans*, *A. niger*)

**Mycobactericidal, including tuberculocidal:**

EN 14348, EN 14563 (*M. terrae*, *M. avium*)

**Virucidal:** EN 14476 (tested on Poliovirus, Adenovirus, including HIV, HBV, HCV, H1N1, H5N1, all human and animal type A influenza viruses)

#### ACTIVE INGREDIENTS

100 g of **ZETA 3 ULTRA** contains:

- 35.4 g of ethanol
- 35 g of isopropanol
- 0.7 g of dimethyl-didecyl-ammonium chloride

### USE

Spray **ZETA 3 ULTRA** over the surfaces and objects to be disinfected to form a continuous film. Leave to work for at least 1 minute, then clean the surfaces/devices with a tissue and dry them off.

### RECOMMENDATIONS

Do not use **ZETA 3 ULTRA** on delicate and alcohol-sensitive surfaces (i.e. plexiglass and dental chairs) for these surfaces use **ZETA 3 FOAM** instead.

\* Before applying, check compatibility of the surfaces to be treated with alcohol-based products; do not treat delicate or alcohol-sensitive surfaces.



### Packaging

C810002 ZETA 1 ULTRA	125 ml bottle
C810000 ZETA 1 ULTRA	1 litre bottle
C810001 ZETA 1 ULTRA	5 litres container with dosage cap
C810011 ZETA 2 SPOREX	900 g container with dosing spoon
C810012 ZETA 2 ENZYME	1.2 kg container with dosing spoon
C810030 ZETA 3 ULTRA	125 ml bottle
C810021 ZETA 3 ULTRA	750 ml bottle with nozzle trigger spray
C810022 ZETA 3 ULTRA	3 litres container with dosage cap
C810029 ZETA 3 SOFT	125 ml bottle
C810023 ZETA 3 SOFT	750 ml bottle with nozzle trigger spray
C810024 ZETA 3 SOFT	5 litres (2x2.5 litres) container with dosage cap
C810032 ZETA 3 SOFT CLASSIC	125 ml bottle
C810027 ZETA 3 SOFT CLASSIC	750 ml bottle with foamer trigger spray
C810028 ZETA 3 SOFT CLASSIC	5 litres (2x2.5 litres) container with dosage cap
C810060 ZETA 3 WIPES	1 tub, 120 wipes each
C810061 ZETA 3 WIPES	1 refill bag, 120 wipes each
C810063 ZETA 3 WIPES TOTAL	1 tub, 120 wipes each
C810062 ZETA 3 WIPES TOTAL	1 refill bag, 120 wipes each
C810025 ZETA 3 FOAM	750 ml bottle with foamer trigger spray
C810026 ZETA 3 FOAM	3 litres container with dosage cap
C810064 ZETA 3 WIPES POP-UP	100 wipes soft pack
C810037 ZETA 4 WASH	3 litres container
C800061 ZETA 5 UNIT	5 litres container

C810044 ZETA 6 HYDRA	125 ml bottle
C810042 ZETA 6 HYDRA	1 litre bottle with dosage cap
C810043 ZETA 6 HYDRA	5 litres container with dosage cap
C810047 ZETA 6 DRYGEL	80 ml bottle
C810045 ZETA 6 DRYGEL	500 ml bottle
C810046 ZETA 6 DRYGEL	1000 ml bottle
C810052 ZETA 7 SOLUTION	125 ml bottle
C810048 ZETA 7 SOLUTION	1 litre bottle
C810049 ZETA 7 SOLUTION	5 litres container with dosage cap
C810053 ZETA 7 SPRAY	125 ml bottle
C810050 ZETA 7 SPRAY	750 ml bottle with foam cap

### Accessories

C800090	1 litre sterilization bath
C800092	3 litres sterilization bath
C810070 ZETA HYGIENE NOZZLE	Dosage cap for 3 and 5 litres container

### BRANCHES

**GERMANY**  
Zhermack GmbH  
Deutschland  
Öhlmühle 10 - D-49448 Marl  
Tel. +49 - 0 54 43 / 20 33 - 0  
Fax +49 - 0 54 43 / 20 33 - 11  
info@zhermack.de  
www.zhermack.com

**POLAND**  
Zhermapol Sp. z o.o.  
ul. Augustówka 14  
02 - 981 Warszawa - Polska  
Tel. +48 - 22 858 82 72  
+48 - 22 858 73 41  
Fax +48 - 22 642 07 14  
biuro@zhermapol.pl  
www.zhermapol.pl

**USA**  
Zhermack Inc.  
P.O. Box 4195, River Edge  
New Jersey 07661 - 4195  
Tel. +1 - (877) 819 6206  
+1 - (732) 389 8540  
Fax +1 - (732) 389 8543  
info@zhermackusa.com  
www.zhermackusa.com

### COUNTRY CONTACTS

**UK AND IRELAND**  
Tel. +44 - (0)7860 346 129  
uk@zhermack.com

**SPAIN**  
Tel. +34 - 900 99 39 52  
info.spain@zhermack.com

**TURKEY**  
Tel. +90 533 76 44 955  
info.turkey@zhermack.com

**FRANCE**  
Tel. +33 - 0800 915 083  
info.france@zhermack.com

**RUSSIA**  
Tel. +7 - 916 930 1191  
info.russia@zhermack.com

**MEXICO AND CENTRAL AMERICA**  
Tel. +52 - 1 55 919 804 97  
info.mexico@zhermack.com

**SOUTH AMERICA**  
Tel. +57 - 310 2601 777  
info.colombia@zhermack.com

### Zhermack SpA

Via Bovazecchino, 100 - 45021 Badia Polesine (RO) ITALY  
Tel. +39 - 0425 597611 Fax +39 - 0425 53596 info@zhermack.com www.zhermack.com